


Hip Fix Mark 1 - Directions for Use

1. Place the patient on the operating table with the affected hip upwards.

2. **Posterior.**


Mount the vertical posterior support on the operating table.
Adjust the support bars so the pad is against the patient's sacrum.

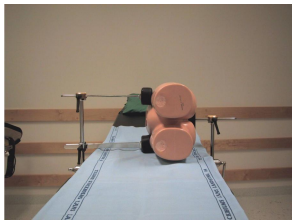

Lock the vertical support bar to the operating table.

3. **Anterior.**

Mount the vertical anterior support on the operating table.
Adjust both anterior supports so they are positioned against the patient's iliac crest.

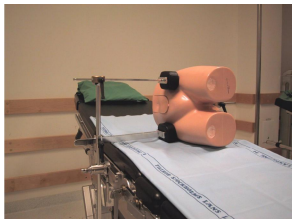


Lock the vertical support bar to the operating table.

Push the lower horizontal support lightly against the operating table.
Lock it with the locking screw.

Adjust the patient so the pelvis is perpendicular to the table.
Lock the upper horizontal support with the locking screw.

4. **Posterior.**

Press the posterior horizontal support against the patient to get stable fixation.

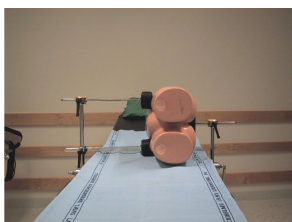

Lock the horizontal posterior support.

Notes

If there is a risk of skin necrosis, upholstery should be applied on the support cushion. Verify that the support cushion and lock handle are fixed in the support arm. If any part of the Hip Fix shows signs of wear, it is essential for achieving the desired results and absolute stability that this or these parts are replaced. Cease use immediately if any parts appear faulty. Check for any wear of the operating table attachments so that they do not negatively affect the use and mounting of Hip Fix. Handles only need to be tightened by HAND force only. Weight limit of patient 150kg.